HOLDING THE SYRINGE

Ergonomic, user friendly handle, perfect balance and smooth plunger travel assure fatigue-free operation even during mass vaccinations at rates of up to 800 injections per hour.

To operate 167 model, place forefinger and middlefinger in the two rings of the syringe handle and move spring-loaded plunger with the thumb.

On 187 and 287 models, place hand on handle and move plunger with the palm of the hand.

To change position of the syringe handle (41), untighten screw (44), turn syringe body to the desired position and retighten screw (44).

The handle of the 20 mL model can be moved forward and backward on socket (3).

OPERATION

All self-refilling syringes are supplied ready to use, no assembling or preparation is required.

• Self-refilling syringe with vial holder

To draw liquid out of an original vaccine bottle with septum closure, remove lock (55), attach vaccine vial (penicillin neck, Ø 20 mm) to intake needle (52), slide lock in again. Rubber washer (1.187.745) improves tightness. Disassemble vial holder by removing lock (55) and ring (54). Vial holder fits models up to 10 mL only.

• Self-refilling syringe with feed tubing

To aspirate liquid from a bottle with rubber stopper carried at the belt, connect one end of tube (23) to the syringe. Puncture bottle cap with feed needle (26) and vent cannula (25). Connect feed tube to the feed needle. When the bottle is hanging, with neck towards the bottom, it is recommended to use the optional draw off needle with air inlet and filter (cat. No. 1.187.590).

To aspirate liquid from an open vial, connect sinker (24) to the end of feed tube (23) and immerse in liquid (note: vent cannula (25) to be used as sinker on 20 mL models).

PP reservoir of 60, 125, or 250 mL capacity (see list of accessories) can be fitted on all syringe types up to 10 mL, after adaptation of a Rotlauf intake valve (Cat. No. 187.613).

Attach injection needle (not supplied) to nozzle (14). Optional manifold (Cat No. 1.287.685) allows injection through one needle on Twin 287 model.

Volume setting

Loosen locking nut (4) and adjust plunger edge to the desired volume graduation on glass barrel by turning socket (3). Tighten locking nut (4) in order to prevent any volume alteration during plunger activation.

On 287 Twin syringe, the volume is independently adjustable on each barrel.

Priming

Prime valve system by activating plunger until liquid fills up the syringe. Set syringes of small capacity on largest volume to prime. To expel remaining air bubbles, hold the syringe upright and give the plunger a few short strokes. The liquid lost during priming may be collected in a clean recipient and returned to the reservoir.

End of dosing

When dosing is completed, remove feed sources and activate plunger several times in order to eliminate any residual liquid. Rinse, disassemble, clean and dry the instrument parts according to the instructions in the "maintenance" chapter. **Never store a syringe without prior priming, cleaning and drying.**

MAINTENANCE

Cleaning

The self-refilling syringe and accessories must be carefully cleaned and dried after each use.

Therefore, aspirate an appropriate cleaning solution (water or detergent) and rinse thoroughly. Activate the syringe several times with air to remove residual liquid before disassembling.

Disassembling

Remove needle, tubing and/or vial holder. Hold handle (41) and unscrew casing (8) in order to access the glass barrel (7). Push rod (1) or handpiece (48) (Twin 287) to free plunger (6). Unscrew aspiration (50) and dispensing valves (14). Dry the valve assembly (seat, ball), syringe body, barrel and plunger before reassembling and storing.

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Trouble shooting

A clogged aspiration valve can easily be fixed up by gently detaching the ball from the valve seat. Therefore, introduce a small pin into the intake valve opening and push until the ball moves. If this operation is not possible, immerse the valve in hot water containing an appropriate detergent and operate the plunger. Make sure the parts are rinsed thoroughly and dried completely.

To prevent a mispositioning of aspiration valve ball, do not release spring loaded plunger rod abruptly while aspirating.

During the distribution, a thin film of liquid between the plunger and the barrel guarantees smooth plunger movement and tightness. After long series of injections, some liquid may be found in the back of the plunger and should be removed when vaccination or dosing is completed.

Only use original spare parts to replace any damaged components of the syringe.

Sterilization

Warning: optional PVC protection sleeve on glass barrel and vial holder rubber washer are not autoclavable. If used, remove them before autoclaving the instrument.

The self-refilling syringe can be autoclaved fully assembled at 121° C for 20 minutes (1 ATM). Do not sterilize an instrument that has not previously been primed and cleaned. Risk of glass breakage during autoclaving is reduced by slightly tightening the syringe body. Retighten of all connections before each use. Correct autoclaving and resulting sterility are the responsibility of the user.

MATERIALS

All parts in contact with the liquid are made of carefully selected, chemically inert materials:

- Feed tubing: silicone
- Barrel: sodocalcic AR glass
- Plunger: stainless steel
- Barrel washer: FPM
- Valve washers: PTFE
- Valve springs and balls: stainless steel
- Sinkers and aspiration cannula: nickel/chrome plated brass.

PERFORMANCE AND ORDERING INFORMATION

Package contents

Standard supply includes Luer Lock nozzle, set of spares for valves (springs, valve balls and washers), operating instructions and according to the model:

- 187.1: vial holder set (vial holder, locking ring, bottle lock, tube connector, feed needle, rubber washer).
- 167.2 and 187.2 : feed tube set (silicone tubing, sinker (except 20 mL model), aspiration and vent cannulas). Twin 287 includes two each of the above accessories.
- 187.3: both sets.
- 187: 2 mL and 5 mL, additional spring for viscous liquids.

Optional threaded Rotlauf and Record Unifix nozzles for models up to 10 mL available on request. Work station to be ordered separately.

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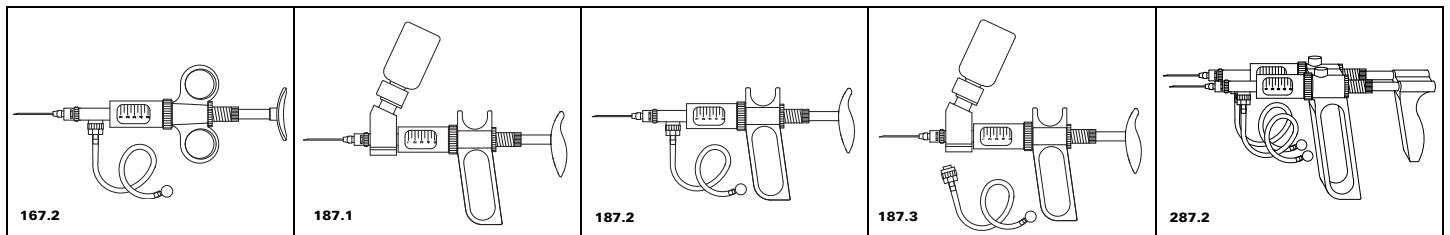
¹⁾ at 0,075 mL

Viscosity data

Volume mL	Maximal cinematic viscosity * (cst)	Example of liquids
0.3	1800	SAE 50 oil
0.5	2500	Fluid honey
1	1200	Silicon oil
2	400 (up to 600) ¹⁾	Glycerol

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Volume mL	Maximal cinematic viscosity * (cst)	Example of liquids
5	150 (up to 200) ¹⁾	SAE 20 oil
10	20	Cream
20	50	Vegetable oil

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Test conditions to determine max. viscosity
Syringe with horizontal feed tubing
Syringe set at maximal volume
Aspiration time < 3 sec.

*Cinematic viscosity in centistokes (cst). Dynamic viscosity in centipoises (cps) obtained by multiplying cinematic viscosity by liquid density (kg/dm³)

¹⁾ When using viscous liquid spring (see accessories)

Accessories

Cat. No.	Description	Packaging
Spare interchangeable glass barrel		
0187.003	0.3 mL	6 /pk
0187.003P	0.3 mL with plunger	6 /pk
0187.005	0.5 mL	6 /pk
0187.005P	0.5 mL with plunger	6 /pk
0187.01	1 mL	6 /pk
0187.01P	1 mL with plunger	6 /pk
0187.02	2 mL	6 /pk
0187.02P	2 mL with plunger	6 /pk
0187.05	5 mL	6 /pk
0187.10	10 mL	6 /pk
0187.20	20 mL	3 /pk
Nylon brush for barrel cleaning		
1.170.301	0.3, 0.5, 1 mL	6 /pk
1.170.305	1, 2, 5 mL	6 /pk
1.170.320	5, 10, 20 mL	3 /pk
1.170.300	set of 2 each	6 /pk
Optional single needle manifold		
1.287.685	For 287 twin syringe	1 pc
Hard PVDF extention tubing		
1.187.20	20 cm	1 pc
1.187.30	30 cm	1 pc

Zubehör

Cat. No.	Description	Packaging
Spare parts for syringe valve		
1.187.72003	0.3 mL	1 bag
1.187.72005	0.5 mL	1 bag
1.187.7201	1 mL	1 bag
1.187.7202	2 mL	1 bag
1.187.7205	5 mL	1 bag
1.187.7210	10 mL	1 bag
1.187.7220	20 mL	1 bag
Conversion set (without valve)		
1.187.1	Straight vial holder	1 pc
1.187.1A	Same with valve	1 pc
1.187.1C	Slant vial holder	1 pc
1.187.1B	Same with valve	1 pc
1.187.2B	Feed tubing	1 pc
Various syringe accessories		
1.187.100	Injection handle	1 pc
1.187.590	Draw off needle	6 /pk
1.187.745	Rubber washer (56)	10 /pk
320.170	Work station	1 pc
187.752	Viscous liquid spring (2 mL)	1 pc
187.753	Viscous liquid spring (5 mL)	1 pc

Accessoires

Cat. No.	Description	Packaging
PVC barrel protection sleeve		
1.170.4005	0.3 / 0.5 mL	6 /pk
1.170.401	1 mL	6 /pk
1.170.402	2 mL	6 /pk
1.170.405	5 mL	6 /pk
1.170.410	10 mL	6 /pk
1.170.420	20 mL	3 /pk
PP reservoir with Rotlauf thread		
1.187.060	60 mL	1 pc
1.187.150	125 mL	1 pc
1.187.250	250 mL	1 pc
Drenchers - mouth & nose cannula		
376.0925S	Straight, 25 mm	3 /pk
376.1251S	Straight, 51 mm	3 /pk
376.16102S	Straight, 102 mm	3 /pk
376.30203S	Straight, 203 mm	2 /pk
376.0925B	Bent, 25 mm	3 /pk
376.1251B	Bent, 51 mm	3 /pk
376.16102B	Bent, 102 mm	2 /pk
376.30203B	Bent, 203 mm	2 /pk

WARRANTY

The self-refilling syringe is guaranteed for a period of one year against any manufacturing or material defect. Cases due to non-respect of manufacturer's instructions are excluded from the warranty. Glass barrel breakage or damages to the plunger due to improper handling or cleaning are not covered by the warranty. Should regular maintenance not eliminate a detected defect, return instrument to the dealer from whom it was purchased, after having obtained a return authorization.

Decontaminate instrument prior to its return.

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SPARE PARTS

- Piston rod
- Spring
- Socket
- Nut
- Cap
- Piston
- Barrel
- Casing
- Spring
- Washer
- Nozzle valve
- Barrel washer
- Valve ball
- Tubing
- Sinker (except 20 mL model)
- Vent cannula
- Feed needle
- Handle
- Screw
- PTFE sleeve
- Hand piece (287)
- Intake valve
- Connecting tube
- Intake needle
- Vial holder body (2 pieces)
- Ring, vial holder
- Lock, vial holder
- Rubber washer

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